**Bedienungsanleitung | Instruction manual | mode d'emploi**

- EN** Instruction manual including safety hints
- DE** Betriebsanweisung mit Sicherheitshinweisen
- FR** Notice d'utilisation avec informations de sécurité
- NL** Gebruiksaanwijzing inclusief veiligheidsstips
- DK** Navodila za uporabo z varnostnimi napotki
- SI** Pred namestitvijo preberite. Preberite letak z varnostnimi navodili.
- IT** Manuale di istruzioni con indicazioni sulla sicurezza
- ES** Manual de instrucciones con consejos de seguridad
- PT** Manual de instruções com conselhos de segurança
- PL** Instrukcja obsługi zawierająca wskazówki dotyczące bezpieczeństwa
- RO** Manual de instrucțiuni, inclusiv sfaturi de siguranță
- CZ** Návod k použití včetně bezpečnostních pokynů

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EN QUICK GUIDE FOR THE USE OF THE CALF LEG SPLINT

PURPOSE OF THE SPLINT:

A calf leg splint is used to stabilize limbs in newborn calves suffering from congenital tendon contracture (Neuromyodysplasia congenita):

- Correct flexural deformities caused by shortened tendons and / or muscles.
- Start treatment immediately after birth.
- Rapid support for the calf to facilitate walking, standing, and milk intake.

REQUIRED MATERIALS:

- Clean calf leg splint with hook-and-loop straps
- In particularly severe cases: bandaging material (e.g. elastic velcro bandage or self-adhesive cohesive bandage) to additionally wrap around the splint

APPLICATION STEPS:

1. PREPARATION

- Separate the calf from the mother to ensure your own safety
- Restrain the calf (if necessary, with the help of a second person)
- Examine the leg for injuries or skin changes

2. APPLYING THE SPLINT

- The splint is applied from the front with the wide opening at the bottom.
- Ensure that it extends from the hoof to just below the joint
- Extend the limb and fix the splint using the hook-and-loop straps
- The splint must be snug, but must not restrict blood circulation

DURATION & MONITORING

- Wearing duration: depending on severity, 14 to 28 days
- Check the fit of the splint daily
 - Examine skin for pressure points
 - Observe the general condition of the calf

In case of inflammation, pressure sores or lack of healing progress contact a veterinarian *Heilfortschritt*

IMPORTANT NOTES

- If unsure, always consult your veterinarian
- Early correction significantly increases the chances of healing
- Daily monitoring of the animals is absolutely essential

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