[Product name] Disposable Nitrile Examination Gloves

[Model] KDN02M (TTC 402383 / TTC 402384 / TTC 402385 / TTC 402386)

[Size] S / M / L / XL

[Features] Latex-free, Powder-free, Protein-free, Ambidextrous, Food Grade, Non-sterile, Single Use, Fingertip Textured, Oil-proof, High Elasticity, Non-toxic, Resistance to Acid

[Product description] Made of nitrile. Sufficient strength and barrier properties. Non-sterile supply, disposable.

[Intended use] It is used to put on doctors' hands for examination or touch examination of patients' condition.

[Usage] Before wearing gloves, remove all ornaments on hands and trim and smooth nails. According to the general standard of surgical disinfection, the hand parts are clear Clean, disinfect and dry. Wear gloves suitable for individual hand size.

Wet hands will make it more difficult to wear.

[Attention]

1. When wearing gloves, do not wear rings or other ornaments, and keep nails trimmed and smooth.

2. Please choose gloves suitable for your own hand size.

3. Do not use damaged gloves.

4. This product is for one-time use only, please destroy it after use.

5. Too high temperature will be very destructive to plastic gloves.

[Storage conditions] Need be stored in a cool, dry, ventilated, non-corrosive dark environment.

[Manufacturer] CRDLIGHT Optoelectronic Technology Co., Ltd.

[Address] Floor 1-5 Building No.7 & Floor 1-4 Building No.5, No.18 Xinyi Road, Jianghai District, Jiangmen City, Guangdong Province, China

[Distributor] GATARIC DOO, Kninska 48, 74400 Derventa, Bosnia and Herzegovina (+387 53 315 000)

[Website] www.gataric.net

1 Please cut your nails before wearing gloves, fingernails that are too long or too sharp can easily damage gloves.

2 Before wearing, please use your finger-pulp to avoid scratching the gloves.

3 When taking off, turn up the gloves at the wrist and take them off to the fingers, do not pull out from the fingertip.

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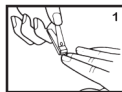
[Proizvođač] CRDLIGHT Optoelectronic Technology Co., Ltd.

[Adresa] Floor 1-5 Building No.7 & Floor 1-4 Building No.5, No.18 Xinyi Road, Jianghai District, Jiangmen City, Guangdong Province, China

[Distributer] GATARIC DOO, Kninska 48, 74400 Derventa, Bosnia and Herzegovina (+387 53 315 000)

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[Hersteller] CRDLIGHT Optoelectronic Technology Co., Ltd.