



## User manual

Digital scales / Model SF-889 Article no. 38129052150

Thank you for purchasing the SF-889 digital scale from our product range.

To ensure proper operation, durability and accuracy of this scale, please read the operating instructions carefully before use.



---

### I. Important product features

- High precision sensor system
- Large LCD display with backlight
- Various units of measurement (kg, lb, b/oz)
- External calibration
- Tare function
- Capacity: 150 kg
- Low battery and overcharge indicator
- Remote display (on the wall, on the desk ...)
- Allows large parcels to be weighed without obscuring the display
- Stainless steel platform
- Power supply: 1.5 V x 4 AA battery or 110 V adapter (included)
- Does not switch off automatically
- Integrated clock (press the ON/TIME button to display the time)
- Minimum weight: 300 g

---

Visit our website

[www.animalcare.horizont.com](http://www.animalcare.horizont.com)

## 2. Operation

- For best results, use the scale on a flat surface.
- Insert the supplied 1.5 V battery into the appropriate compartment or connect the mains adapter.
- Press the "ON/TIME" button to switch the scale on.
- Wait until "0.0" (zero) appears on the display before placing an object on the scale.
- Carefully place the desired item on the weighing platform.
- It will take a few seconds for the weight to stabilize and appear on the LCD display.
- To choose between the different units of measurement (lb, kg, lb/oz), press the "UNIT" button.

## 3. Tare function

The tare button resets the display to 0 when the weight is on the scale. If the tare button is pressed, the scale subtracts the weighed weight. Place the container you wish to use on the scale. When the display has stabilized, press the tare button once. The display is reset to zero so that you can add items to the container and the corresponding net weight is displayed.

## 4. Technical data

- Model: FS-7704
- Capacity: 100 kg / 120 kg / 150 kg / 200 kg
- Resolution: 0.1lb / 0.05 kg / 20 zor 0.21b / 0.1kg / 4oz
- Weight units: Pounds (lb), kilograms (kg), pounds and ounces (lb / oz).
- White LCD screen with backlight
- Dimensions: 392 x 305 x 29 mm
- Power supply: 4AA or mains adapter (both included)
- Minimum weight: 0.3 kg

## 5. Calibration

- Place the scale in clock mode on a flat surface.
- Press the "TARE" button and while holding it down, also press the "ON/TIME" button for about three seconds.
- After three seconds, a random number will appear.
- Press "TARE" again - the display will now show "50 kg". Make sure that the scale is stable and then place the weight of 50 kg on the platform of the scale.
- The display will now show "PASS".
- Calibration is complete.

## **6. Maintenance and care**

- Store the scale in a place protected from moisture
- Do not drop the scale
- Avoid overloading the scale
- Do not leave weights on the scale when it is not in use
- Replace the battery when the message “Lo” appears on the display of the scale

## **7. Attaching the scale to the wall**

1. Connect the scale to the electronic display using the power cable
2. Attach the dial with the enclosed fixing screws
3. Insert the mains plug into the socket to supply the battery